



Vertical Folding Earloop Protective Mask

USER INSTRUCTIONS:


1. Please read the instructions provided in this manual **carefully**. Any deviation from the instructions during times of exposure can reduce the overall effectiveness of the protective mask which may result in sickness or death.
2. Before occupational use of this protective mask, a written respiratory protection program must be implemented meeting **all** the local government requirements.
3. Leave the contaminated area **immediately** if you experience any sign of dizziness, irritation, or other issues.
4. When not in use, store the protective mask away from contaminated areas.
5. Discard used protective masks after one use. Disposal of the protective mask must be in accordance with **all** applicable regulations.
6. Do **NOT** attempt to modify or use the protective mask in ways other than instructed.
7. Do **NOT** use the protective mask with beards or facial hair as it may prevent a good facial seal.
8. Discard the protective mask and **replace** with a new one if:
 - Excessive **clogging** of the protective mask causes breathing difficulty.
 - The protective mask becomes damaged.
9. Do **NOT** use the protective mask or enter or stay in a contaminated area under the following circumstances:
 - Atmosphere contains **oil aerosols**.
 - Atmosphere contains **less than 19.5% oxygen**.
 - For protection against gases or vapors.
 - Contaminants or their concentrations are unknown or **immediately dangerous to life or health**.
 - For sandblasting, paint-spray operations, asbestos.
10. **DO NOT** use this mask in a clinical setting where the infection risk **level** through inhalation exposure is high.

STORAGE INSTRUCTIONS

 **End of Shelf Life** Use the protective mask before the “use by” date

 **Storage Temperature Range** -4°F to 86°F [-20°C to 30°C].

 **Storage Maximum Relative Humidity** <80% RH

 **Dimensions** 4.17” x 6.14” [106mm x 156mm]

FITTING INSTRUCTIONS:

E95
Vertical Folding Earloop Style



STEP 1

Unfold the mask and allow the earloop bands to hang outward on both sides.



STEP 2

Position the mask on your face with the adjustable metal nosepiece facing up.



STEP 3

Pull the earloop bands around both ears. Then, adjust for fit and comfort.



STEP 4

Using both hands, press down on the metal nosepiece to form a tight-fitting seal around your nose.



STEP 5

Test the seal by cupping both hands over the protective mask. Then, begin to inhale and exhale sharply. If air leaks around the mask's edges, adjust the straps and reposition the nose piece until a proper seal is formed.

PERFORMING A SEAL CHECK:

It is necessary to perform a seal check on the protective mask every time you wear it.

Cup both hands around the protective mask and inhale and exhale forcefully several times. If you notice any air leakage, adjust the nosepiece and earloop bands until the leakage is corrected. Once corrected, repeat the seal check again.

! **WARNING** The protective mask should not be used without a proper fit. This protective mask helps protect against certain contaminants.

Materials which may come into contact with the wearer's skin could cause allergic reactions to susceptible individuals.

Learn more at **BNX.com**



Manufactured by
BNX Converting, LLC.

16727 Park Row, Houston, TX 77084, USA

